


CovaWound™ Hydrocolloid Thin/Transparent

Hydrocolloid Dressing

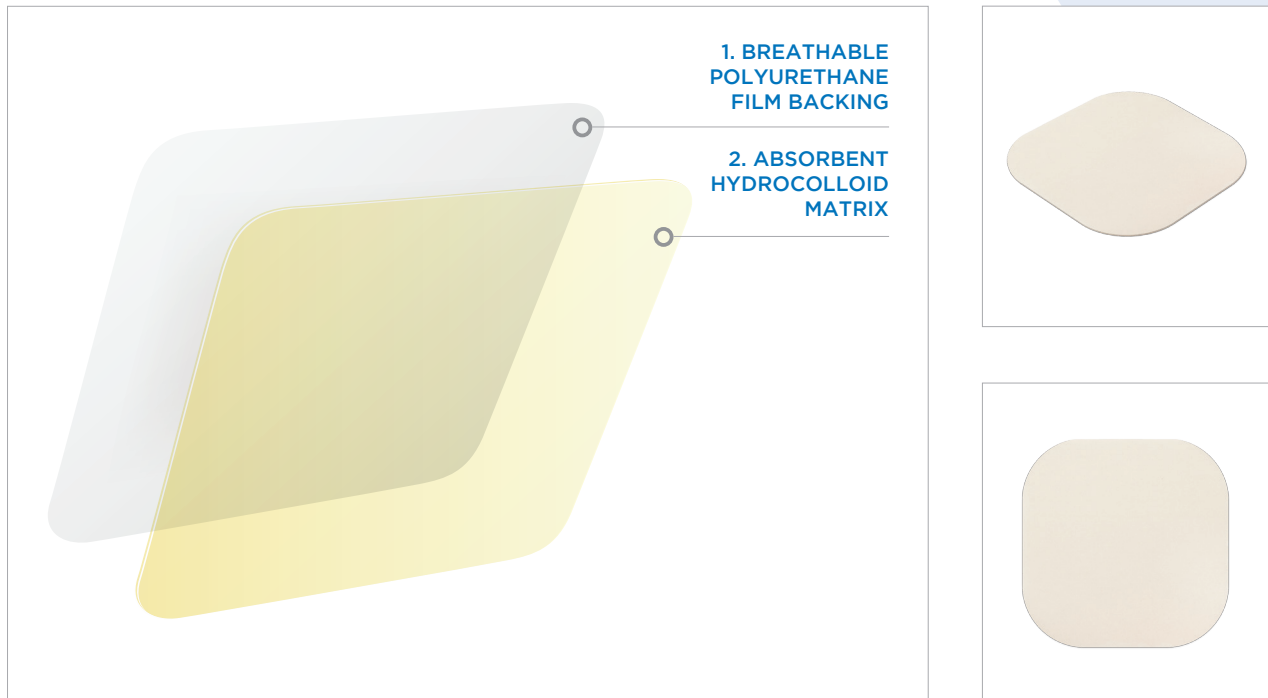

An absorbent hydrocolloid matrix which has been laminated on to a breathable polyurethane film backing.

The hydrocolloid matrix absorbs exudate to form a cohesive gel whilst the breathable polyurethane film provides a high rate of moisture vapour transmission. The combination of these features ensures that CovaWound™ Hydrocolloid provides the optimal moist wound healing environment.

A two-layer constructed dressing

1. BREATHABLE POLYURETHANE FILM BACKING

Provides a high rate of moisture vapour transmission. Acts as a barrier to external contamination.

2. ABSORBENT HYDROCOLLOID MATRIX

Absorbs exudate to form a cohesive gel that creates a moist wound environment.

Exudate level

👉👉👉 Light

Exudate level indicates the highest level of exudate that the dressing can accommodate.

Product Codes and Sizes: Available to order through NHS Supply Chain Advanced Wound Care Framework and Alliance Healthcare

Dressing Size (cm)	Pack Size	Covalon Article Number	PIP Code	Catalogue Code
5 x 10	10	5200510		ELM518
5 x 20	10	5200520		ELM519
10 x 10	10	5201010	406-1057	ELM520
10 x 15	10	5201015	406-1032	ELM516
10 x 25	10	5201025		ELM517
15 x 15	10	5201515	406-1040	ELM521
15 x 20	10	5201520		ELM522

Indications for Use

CovaWound™ Hydrocolloid dressing is indicated for the management of lightly exuding wounds such as pressure, leg and foot ulcers, superficial partial-thickness burns, donor sites, diabetic ulcers, post-operative wounds and skin abrasions. CovaWound™ Hydrocolloid dressing may also be used on small wounds towards the end of the healing phase.

Application

- Clean the wound area with normal saline or distilled water.
- Gently dry the surrounding skin and ensure that it is free of any residue.
- Choose the size and shape of the dressing which is most suitable, the dressing should extend 2 cm beyond the wound margin.
- Remove the large piece of the release liner from the dressing.
- Centre and apply the exposed part of the dressing over the wound bed.
- Avoid touching the adhesive surface to minimize risk of contamination.
- Remove the small piece of the release liner from the dressing and apply.
- Finalize application of the dressing, making sure that the edges of the dressing adhere well. This initial adhesion will improve as the dressing warms up. If necessary, use medical tape around the edges to avoid roll-up.

CovaWound™ Advanced Wound Care Line




CovaWound™
Silicone & Silicone Lite



CovaWound™
Silicone & Silicone Lite



CovaWound™
Alginate

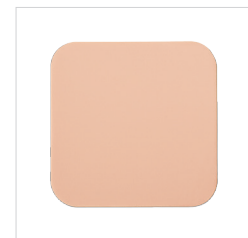

CovaWound™
Foam



CovaWound™
Foam



CovaWound™
Hydrocolloid &
Hydrocolloid Thin /
Transparent



CovaWound™
SuperAbsorbent



MediClear™
Scar

Covalon Technologies (Europe) Ltd
Lion Court, Staunton Harold Hall,
Melbourne Road, Ashby-de-la-Zouch LE65 1RT
+44 (0)1332 695020 www.covalon.co.uk

CW520-UK-3 (4/2018)

covalon
TECHNOLOGIES LTD.

CovaWound is a trademark of Covalon Technologies Ltd.