


CovaWound™ Hydrocolloid

Hydrocolloid Dressing with Border

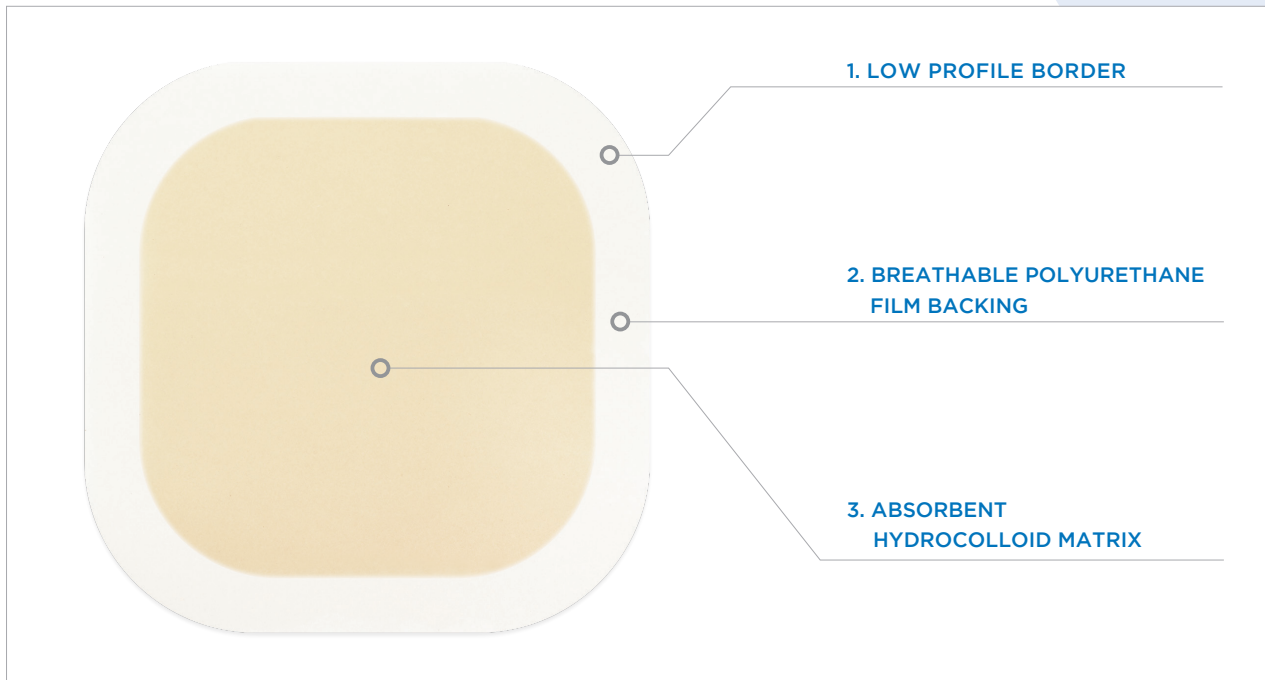

CovaWound™ Hydrocolloid Dressing with Border is composed of an absorbent hydrocolloid matrix that has a hydrocolloid border laminated onto a breathable polyurethane film backing. The hydrocolloid matrix rapidly absorbs exudate to form a cohesive gel, while the breathable polyurethane film backing provides a consistently high rate of moisture vapour transmission.

A three-layer constructed dressing with border

1. LOW PROFILE BORDER

Improves adhesion and reduces risk of edge roll.

2. BREATHABLE POLYURETHANE FILM BACKING

Provides a high rate of moisture vapour transmission. Acts as a barrier to external contamination.

3. ABSORBENT HYDROCOLLOID MATRIX

Absorbs exudate to form a cohesive gel to create a moist wound environment.

Exudate level



Exudate level indicates the highest level of exudate that the dressing can accommodate.

Product Codes and Sizes: Available to order through NHS Supply Chain Advanced Wound Care Framework and Alliance Healthcare

Dressing Size (cm)	Pack Size	Covalon Article Number	PIP Code	Catalogue Code
5 x 5	10	5010505	406-5769	ELM512
10 x 10	10	5011010	406-5843	ELM510
15 x 15	10	5011515	406-5751	ELM511
10 x 12	10	5011012 (Oval)		ELM509

Indications for Use

CovaWound™ Hydrocolloid dressing is indicated for the management of lightly exuding wounds such as pressure, leg and foot ulcers, superficial partial-thickness burns, donor sites, diabetic ulcers, post-operative wounds and skin abrasions. CovaWound™ Hydrocolloid dressing may also be used on small wounds towards the end of the healing phase.

Application

- Clean the wound area with normal saline or distilled water.
- Gently dry the surrounding skin and ensure that it is free of any residue.
- Choose the size and shape of the dressing which is most suitable, the dressing should extend 2 cm beyond the wound margin.
- Remove the large piece of the release liner from the dressing.
- Centre and apply the exposed part of the dressing over the wound bed.
- Avoid touching the adhesive surface to minimize risk of contamination.
- Remove the small piece of the release liner from the dressing and apply.
- Finalize application of the dressing, making sure that the edges of the dressing adhere well. This initial adhesion will improve as the dressing warms up. If necessary, use medical tape around the edges to avoid roll-up.

CovaWound™ Advanced Wound Care Line



CovaWound™
Silicone & Silicone Lite



CovaWound™
Silicone & Silicone Lite



CovaWound™
Alginate



CovaWound™
Foam



CovaWound™
Foam



CovaWound™
Hydrocolloid &
Hydrocolloid Thin /
Transparent



CovaWound™
SuperAbsorbent



MediClear™
Scar

Covalon Technologies (Europe) Ltd
Lion Court, Staunton Harold Hall,
Melbourne Road, Ashby-de-la-Zouch LE65 1RT
+44 (0)1332 695020 www.covalon.co.uk

CW501-UK-3 (4/2018)

covalon
TECHNOLOGIES LTD.

CovaWound is a trademark of Covalon Technologies Ltd.