


# CovaWound™ Foam

## Non-Adherent Foam Dressing

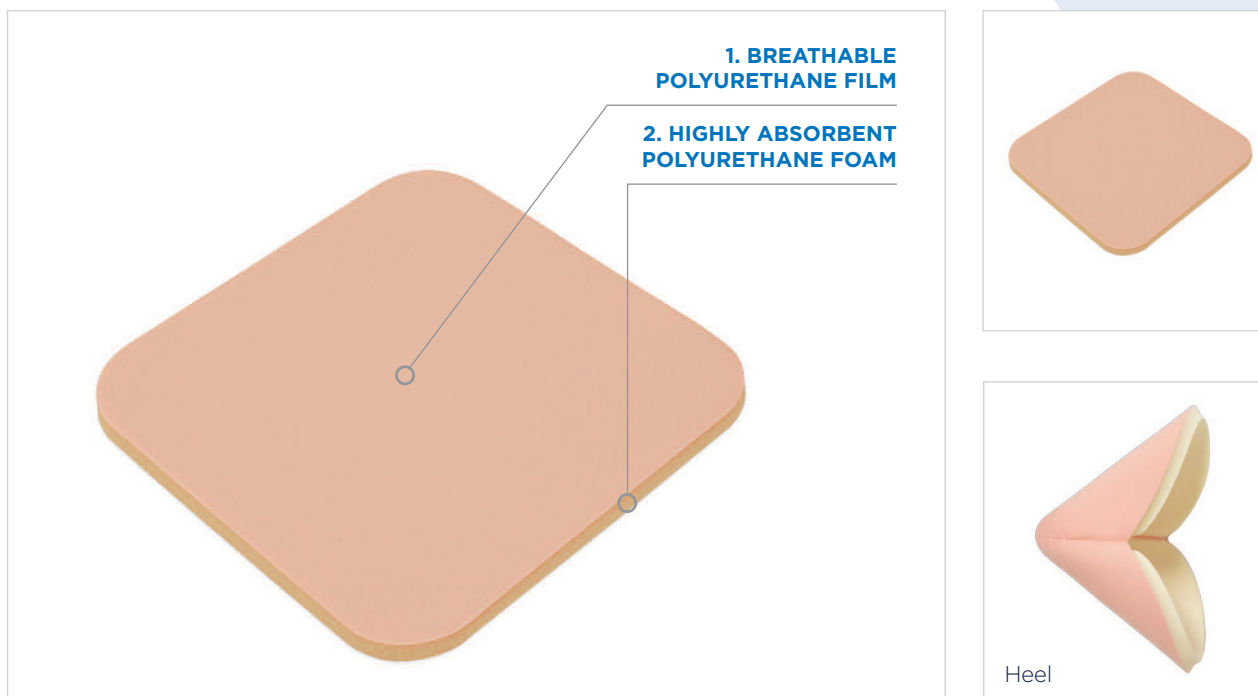

A non-adherent foam dressing with a two layer construction composed of highly absorbent polyurethane foam with a breathable polyurethane film backing. The absorbent polyurethane foam core is designed to absorb and retain exudate, while the breathable outer layer allows moisture vapour transmission away from the dressing. CovaWound™ Foam dressing enables healing to progress by creating and maintaining a moist healing environment. The breathable pink outer layer protects against eschar formation and is also waterproof, protecting against external contamination.

CovaWound™ Foam dressing is soft and conformable providing an optimal moist healing environment and excellent patient comfort. As CovaWound™ Foam dressing absorbs exudate, it may swell slightly but will not leave debris in the wound.

### 1. BREATHABLE POLYURETHANE FILM BACKING

Allows moisture vapour transmission away from the dressing. Designed to protect against eschar formation and is also waterproof, protecting against external contamination.

### 2. HIGHLY ABSORBENT POLYURETHANE FOAM

Designed to absorb and retain exudate, while still maintaining a moist wound environment.

#### Exudate level

👉 👉 👉 Heavy

Exudate level indicates the highest level of exudate that the dressing can accommodate.

# Product Codes and Sizes: Available to order through NHS Supply Chain Advanced Wound Care Framework and Alliance Healthcare

Dressing Size (cm)	Pack Size	Covalon Article Number	PIP Code	Catalogue Code
5 x 5	10	2000505	406-5736	ELA959
10 x 10	10	2001010	406-5850	ELA938
15 x 15	10	2001515	406-5835	ELA955
20 x 20	10	2002020	406-5819	ELA957
10 x 13	5	2001013 (Heel)	406-5777	ELM506

## Indications for Use

CovaWound™ Foam dressing does not adhere to the wound and, therefore, is recommended for wounds surrounded by sensitive skin, such as venous ulcers. It is indicated for the management of moderate to heavily exuding wounds such as pressure, leg, and diabetic foot ulcers, superficial and partial-thickness burns, donor sites, trauma wounds and post-operative surgical wounds.

## Application

- Clean the wound area with normal saline or distilled water.
- Gently dry the surrounding skin and ensure that it is free of any residue.
- Choose the size and shape of the dressing which is most suitable, the dressing should extend 2 cm (3/4") beyond the wound margin.
- Remove the dressing from the packaging.
- Remove the release liner from the dressing and apply the white side of the non-adherent foam dressing to the wound.
- A retention method such as taping onto intact skin or bandaging in place with an appropriate bandage or a film dressing is acceptable.

## CovaWound™ Advanced Wound Care Range



**CovaWound™**  
Silicone Border &  
Silicone Border Lite




**CovaWound™**  
Silicone & Silicone Lite



**CovaWound™**  
Sacral



**CovaWound™**  
Alginate

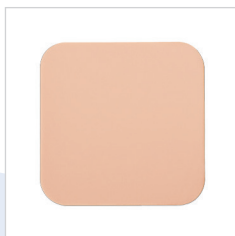

**CovaWound™**  
Foam



**CovaWound™**  
Adhesive Foam



**CovaWound™**  
Hydrocolloid &  
Hydrocolloid Thin/  
Transparent



**CovaWound™**  
SuperAbsorbent